

# Aqua Sonic

The Newest wonderful peeling for Skin Resurfacing

- ┌ Non-Invasive
- 
- └ Painless
- 
- ┌ Multifunctional
- └



## Technical Specifications

Ultrasonic	1.2 MHz
Motor	upto 600mmHg (±10%)
Power consumption	80Watt
Input Voltage	220VAC / 50 / 60Hz
Dimensions	350(D) X 350(L) X 1000 (H) mm
Weight	30Kg
Aqua Handpiece	Face(Regular Size) Nose(Small Size)
Aqua Waste tank	1200cc



ADDRESS : RM. 1412, SJ TECHNO VILLE, 278, BEOTKKOT-RO, GEUMCHEON-GU, SEOUL, KOREA  
Tel 82-2-3281-9091 Fax 82-2-3281-8011 E-mail amimed@amimed.co.kr



Skin Refreshing!  
Make your skin cleaner and whiter.

## Hydro Peeling System Aqua Sonic

Remove horny substances and supply moisture at the same time.

Aqua Sonic is designed to promote exfoliation to remove aged dead skin cells and sebum the same time, injects highly concentrated nutritious serum to make skin and scalp moist, smooth, and lustrous.

Safe, no downtime, no side effect.

Because the treatment does not cause any damage to skin and patients can lead a normal life just after treatment. Also there is no side effect such as injury, pain, burns, rash, etc.

Multiple skincare effects.

Treatment removes foreign materials such as cosmetics and sebum. Therefore, it is expected to achieve whitening and pore-tightening effects.



# Aqua Sonic is ...

Not a decortication old style machine. It is a new-concept natural spa skincare system, which makes your skin exfoliation with no pain. Also, a range of treatment is variety such as cleansing of skin and removal of obsolescent horny substances. If you use Laser Doctor as a helper, it makes your skin nutritious, calm and Epidermal Growth Factor (EGF). After all the treatment, you will feel moist and refreshing. Ultra Sound can release pain, tighten pores and treat for fat.

## Name and function of parts and components

Aqua Sonic is designed to promote exfoliation to remove aged dead skin cells, sebum and hydrate your skin. Laser Doctor as a helper makes your skin nutritious, calm and Epidermal Growth Factor (EGF).


Because treatment does not cause any damage to skin and scalp, patients can lead a normal life right after treatment. There are no side effects such as injuries, pain, burns, rashes, etc.

Treatment removes foreign materials (such as cosmetics and sebum) so users can expect to achieve whitening and pore-tightening effects.



## Special Tips

For removal of keratin as well as for extrusion

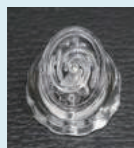

Whole face



Diamond Tips

The special tip of the Aqua Sonic consists of two suction holes and one serum sidcharge hole. Its specially-designed spiral tip is intended to hold a product long enough to soften dead cells of skin, provide nutrients, and absorb them in a vacuum state to maximize its peeling and extrusion effects.

Around the nose

## User Manual for Aqua Sonic

Aqua Sonic adopts a vacuum extrusion method. The hand piece should come into close contact with the skin. You can control pressure for both fine bubbles and serum liquid to perform treatment. Red areas on the skin that may be created during operation will disappear normally in 20~30 minutes and the skin restores healthy and lustrous condition.

- 01 **Cleansing** Wipe wastes 2~3 times off the surface of the face with a cleansing cream
- 02 **Diagnosis of skin and Selection of serum** Identify whether the skin is normal,oily, sensitive, or of acne type and make selection of a serum suitable for the skin type. [Aqua PE:For all skin type]
- 03 **Treatment [removal of keratin and sebum]** First spread the selected fine bubble and serum over the face and then divide the face into 3~5 areas to start treatment for about 2 minutes.
- 04 **Washing** Wash and wipe out the face cleanly with a spongy after treatment.
- 05 **Treatment (supply of moisture and nutrients) and finish** Perform treatment while letting serum penetrate into the skin; after treatment, give a finishing touch with a skin balm or a nourishing cream.



In the order of Neck >>> Chin >>>> Cheek >>>>> Forehead >>>>> Nose

- 1. Neck: after turning face a little sideways, make lymph nodes relax by massaging the area from top to bottom and in the opposite way with the sternocleidomastoid region in the center.
- 2. Nose: perform treatments repeatedly because there is lots of sebum in this region.
- 3. Perform treatment by traveling all across the face from bottom to top along the texture of muscles.
- 4. Treatment is recommended to be made for about 2 minutes respectively in the order of the neck → chin → cheek → forehead → nose.

## Aqua Sonic Solution



Aqua PE Dr. Serum RF Cream for Lifting and Slimming

**AQUA PE [For all skin Type -For the general skin, 30ml]**

Since glucosamine and lactic acids promote the peeling of the skin to remove impure substances softly and plant extracts soothe and moisten the skin smoothly, the product enables a scientific treatment that does not produce any injuries or rashes while achieve a bright and healthy-looking skin regeneration.

**Dr. Serum [Nutrition and supply of moisture for all type of skin, 20ml]**

The deep penetration of vitamin C, E, Aloe Vera, and green tea extract will help skin to restore and regenerate.

**RF Cream [Lifting and Slimming]**

This cream is formulated to use with High Frequency equipment. It is well rubbed until the treatment finish. It helps grow and maintain healthy skin with the botanical extract and anti-oxidant ingredients such as Allantoin, Green Tea Extract, Portulaca Extract, Belamcanda Chinese Water Extract, Tocopheryl Acetate(anti-oxidation ingredients) and Beta Glucan.